



Support Strap Washing/Laundry Instructions

Product	Maximum Water Temperature	Dryer Use	Fabric Composition
VitalGo Support Straps (Blue, w/Velcro)	161°F (72°C)	Low Tumble Dry/No Heat or Drip Dry	Nylon Strap / Silicone Velcro

1. Recommended to wash the straps by themselves.
2. Attach (white) Velcro material (completely and flat) to (blue) Nylon fabric.
3. Set temperature to approved temperature (above) or less.
4. Normal Detergents can be used in the washing process. Bleach can be added to wash at 150 to 300ppm.
5. Air dry or tumble dry no heat.
6. Straps should not be ironed.
7. Visually inspect straps for unusual wear and tear. If Velcro hooks are missing or seams undone discontinue use of straps and replace with new ones.
8. It is recommended to replace Straps after 25 wash cycles or when visual damage is apparent.

You can find a copy of these or updated instructions on vitalgosys.com/downloads or contact VitalGo via phone at Country code 001 877-678-6781