

TLB425T/V – Uncrating instructions

Tools - Drill with Philips bit
Socket wrench + socket 17mm

Step 1 - Unscrew all the screws to release the top cover.
Follow the red line (screws on upper plate)
and Blue line on the sides.



Step 2 - Take out cover and put aside
You will need it later (stage 13)



Step 3 - Unscrew along the purple line and bottom connections.

Step 4 - Remove remaining screws
connecting side panel to palette
and remove side panel



Step 5 - Remove Nylon off bed

Step 6 - Make sure all plugs are in place and
plastic cover is on.



Step 7 - Connect Power -



Step 8 - Place Scale display gooseneck in place and connect head board

Step 9 - Lift all four side rails to upright position.



TLB425T/V – Uncrating instructions

IMPORTANT !!!

Step 10 - Reset Bed

- A. Hold buttons 3+4 **TOGETHER**. A series of beeps will be heard. Hold until beeping stops.
- B. Hold Buttons 1 + 2 Together - Bed will start moving. Make sure first movement is UP. Wait until foot board goes all the way out and two beeps are heard.



Bed after finishing RESET



IMPORTANT !!!

Step 11- Unscrew bolts connecting bed to pallet. Use Socket 17mm or 14mm

There are four, one in each corner



Step 12 - Activate Directional lock



Step 13 - Bring Bed to caster position by pressing button 9 on the handset. Wait until two beeps are heard, indicating casters are all the way down

Step -14 Take top cover and place at end of pallet on foot side



Step -15 Move bed off pallet



TLB425T/V – Uncrating instructions

Done !

