

VitalGo Total Lift Bed V5 Set-Up Checklist

***Please complete ALL of the following steps in order prior to placing the patient on the bed.**

1. Plug the bed into a working wall outlet
 - a. Ensure green light on 3rd row of the hand pendant (power indicator) is illuminated.
2. Complete the Initialization sequence
 - a. **PRESS** and **CONTINUOUSLY** hold buttons 3+4 (Raise/Lower Legs) without interruption on the hand pendant or Head of Bed side rail for ~ 15 seconds.
 - b. **PRESS** and **CONTINUOUSLY** hold buttons 1+2 (Raise/Lower Head) without interruption on the hand pendant or Head of Bed side rail until bed completes all movements and double beep is audible.
 - c. Return the bed to flat position using button 11 (CPR/Flat button) on the hand pendant or Head of Bed side rail.
3. Power on the mattress (if not already on).
 - a. Ensure CPR valve located at the Head of Bed is **CLOSED**.
 - b. Set mattress settings to desired settings.
 - i. Refer to Total Lift Bed Therapeutic Surface reference handout for further instruction if needed.
 - c. **LOCK** mattress setting control panel once desired settings selected.
4. Zero the Bed Scale (must be on stationary frame/wheels up)
 - a. Turn the Scale Box **ON** and ensure **ALL** necessary linens and equipment are placed on bed.
 - b. Zero Bed Scale by **PRESSING** and **HOLDING** button 3 (Total Weight/Bed Zero) for 3 seconds on the Scale Box.
 - c. Zero Footboard Scale by **PRESSING** and **HOLDING** button 5 (Foot Board/Foot Board Zero) for 3 seconds on the Scale Box.
 - i. Ensure footboard is extended beyond the length of the mattress for accurate weight/biofeedback during the zeroing process.
 - d. Any additional items (IV Pole, O2 Tank and Holder, SCD's, etc) placed on the bed frame after calibration will be reflected on the total weight.
5. Transfer patient to the Total Lift Bed