


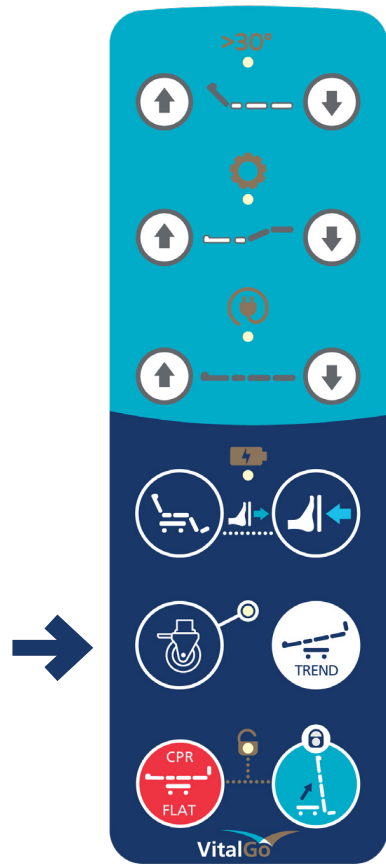

After manually lowering Bed function cannot be operated until the following steps are taken-

1. Press and hold the Wheel Button (button # 9)
Short beeps will be heard, continue pressing until the wheels reach the floor and beeping stops.
2. Replace the TILT CPR Safety Pin with the red CPR Tag (figure 2) .

Bed can now be operated normally.



After manually lowering Bed function cannot be operated until the following steps are taken-

1. Press and hold the Wheel Button (button # 9)
Short beeps will be heard, continue pressing until the wheels reach the floor and beeping stops.
2. Replace the TILT CPR Safety Pin with the red CPR Tag (figure 2) .

Bed can now be operated normally.

Emergency and Manual Tilt Lowering

In the event that the patient needs to be lowered quickly from the tilt position, or in case of power or electronic failure, follow instructions for manual emergency Quick Release lowering from the tilted position and bed position reset:

1. Remove Safety Pin by pulling the TILT CPR tag, at the lower head side of the bed (Figure 1&2).
2. Pull the red lever (Figure 3, at top of bed under headboard pole) towards outside of the bed. Tilt frame will start moving down. Movement will stop when lever is released or support surface is horizontal.

Ensure all persons and body parts are clear of the bed during manual lowering.

Remark -

If the bed is in full tilt and does not begin to lower, gently pull the top of the support surface to initiate the movement.



Figure 1

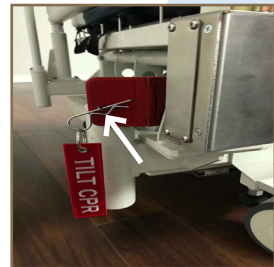


Figure 2

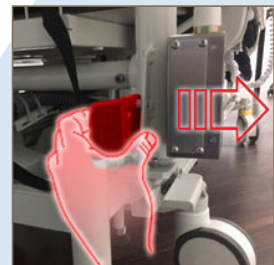


Figure 3

(Turn to other side)

Emergency and Manual Tilt Lowering

In the event that the patient needs to be lowered quickly from the tilt position, or in case of power or electronic failure, follow instructions for manual emergency Quick Release lowering from the tilted position and bed position reset:

1. Remove Safety Pin by pulling the TILT CPR tag, at the lower head side of the bed (Figure 1&2).
2. Pull the red lever (Figure 3, at top of bed under headboard pole) towards outside of the bed. Tilt frame will start moving down. Movement will stop when lever is released or support surface is horizontal.

Ensure all persons and body parts are clear of the bed during manual lowering.

Remark -

If the bed is in full tilt and does not begin to lower, gently pull the top of the support surface to initiate the movement.



Figure 1



Figure 2

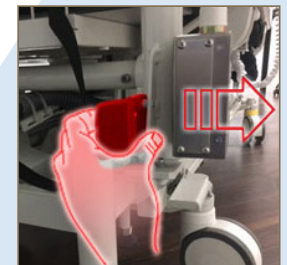


Figure 3

(Turn to other side)