

### C. Foot Lifter Scale

The Foot-Lifter scale reading can be seen in two displays, the display on the Foot-lifter and the main display.

The Foot-lifter display will always show the amount of pressure placed on the Foot-Lifter. For the main display –

1. Press “Foot Board” (button “5”). The pressure on the Foot-Lifter will be displayed.
2. Pressing the button a second time gives the pressure on the Foot -Lifter as a percentage of the patient weight. An arrow will appear on the display next to the % (number “7” on the diagram).
3. To “ZERO” the Foot-Lifter scale –
  - Press “Foot Board” (button “5”).
  - Press Button “5” again for 4 seconds.

**IMPORTANT** – Always “ZERO” when the bed is in the horizontal position.

### D. Weigh Bearing

WEIGH BEARING IS THE PERCENTAGE OF THE PRESSURE APPLIED BY THE PATIENT’S FEET ON THE FOOT-LIFTER IN RELATION TO HIS WEIGHT.

1. When the display is showing the patient weight, press “Foot-Board” button “5” – the main display will show the amount of pressure on the footboard.
2. Press again button “Foot-Board” (button “5”) – the display will show the percentage of the pressure on the Foot-Lifter to the patient weight. The arrow on the % on the top right side of the display will show “7”.
3. To return to patient weight press “Total Weigh” (button “3”) short press.

### E. Alarm Feature

1. Press Clear/Set Alarm (button “4”) until two beeps are heard.
2. An arrow will appear next to the words Alarm Set (number “8”) on the diagram. Alarm is now armed.
3. If patient exits the bed (or items are removed from the bed) the buzzer will sound.

**NOTE** – If scale is in “HOLD” mode too long (see “B” above), the buzzer will sound and the display will flash “HOLD”.

**TO SHUT ALARM** – press “Clear Alarm” (button “4”) quick press or “WEIGH” (button “2”) if in the “HOLD” mode.

### C. Foot Lifter Scale

The Foot-Lifter scale reading can be seen in two displays, the display on the Foot-lifter and the main display.

The Foot-lifter display will always show the amount of pressure placed on the Foot-Lifter. For the main display –

1. Press “Foot Board” (button “5”). The pressure on the Foot-Lifter will be displayed.
2. Pressing the button a second time gives the pressure on the Foot -Lifter as a percentage of the patient weight. An arrow will appear on the display next to the % (number “7” on the diagram).
3. To “ZERO” the Foot-Lifter scale –
  - Press “Foot Board” (button “5”).
  - Press Button “5” again for 4 seconds.

**IMPORTANT** – Always “ZERO” when the bed is in the horizontal position.

### D. Weigh Bearing

WEIGH BEARING IS THE PERCENTAGE OF THE PRESSURE APPLIED BY THE PATIENT’S FEET ON THE FOOT-LIFTER IN RELATION TO HIS WEIGHT.

1. When the display is showing the patient weight, press “Foot-Board” button “5” – the main display will show the amount of pressure on the footboard.
2. Press again button “Foot-Board” (button “5”) – the display will show the percentage of the pressure on the Foot-Lifter to the patient weight. The arrow on the % on the top right side of the display will show “7”.
3. To return to patient weight press “Total Weigh” (button “3”) short press.

### E. Alarm Feature


1. Press Clear/Set Alarm (button “4”) until two beeps are heard.
2. An arrow will appear next to the words Alarm Set (number “8”) on the diagram. Alarm is now armed.
3. If patient exits the bed (or items are removed from the bed) the buzzer will sound.

**NOTE** – If scale is in “HOLD” mode too long (see “B” above), the buzzer will sound and the display will flash “HOLD”.

**TO SHUT ALARM** – press “Clear Alarm” (button “4”) quick press or “WEIGH” (button “2”) if in the “HOLD” mode.

# Total-Lift™ Bed Weight Bearing Control System

(For detailed instructions, safety guidelines and warnings please refer to the owner's manual)

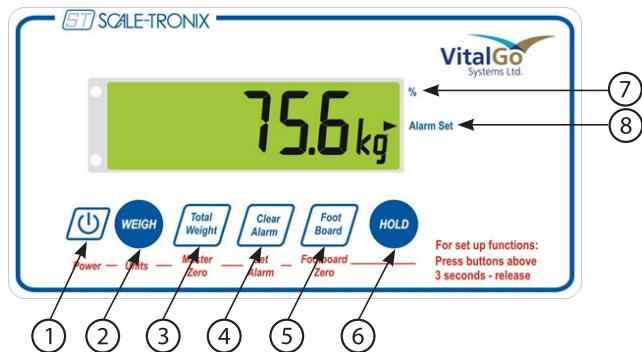

**IMPORTANT** – Scale must be turned on when bed is in operation, otherwise bed will not function well.

## OPERATION –

### A. For patient weight

1. If display is not “On”, press On/Off (button “1”) to start scale operation.
2. Bring the bed to horizontal position using the CPR button on the handset (button 9). Check that the required bedding is on the bed, and the patient is not on the bed.
3. Master Zero – Press and hold Master Zero (button “3”) for 4 seconds and release after hearing two beeps.  
**Important Note** - Do Not activate Master Zero - button ‘3’ when patient is in bed.
4. Scale will read 0.0 (Zero) – patient can then be placed on the bed.
5. The weight of the patient will be displayed after a few seconds.
6. After scale is not active for 10 minutes it will display “SCALE”. To show weight again, press “Weigh” button “2” or “Hold” button “6”.
7. Kg/ Lb – to switch between Kg/Lb press “WEIGH” (button “2”) for 3 seconds.


### B. Adding / Removing bedding / items to the bed

1. Press “HOLD (button “6”).
2. Add items to the bed.
3. Press “WEIGH” (button “2”) and the weighing feature will resume, not changing the patient weight reading.
4. To remove items from the bed, follow same procedure.

(Turn to other side)

# Total-Lift™ Bed Weight Bearing Control System

(For detailed instructions, safety guidelines and warnings please refer to the owner's manual)

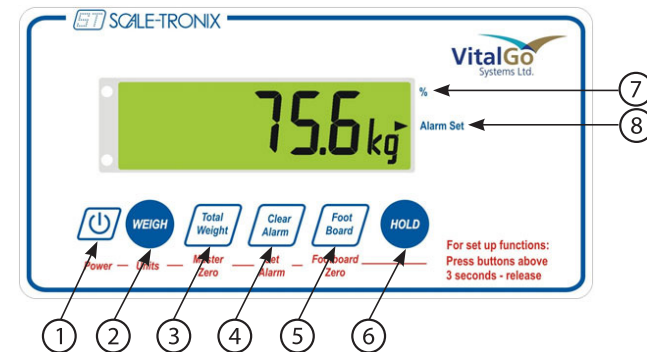

**IMPORTANT** – Scale must be turned on when bed is in operation, otherwise bed will not function well.

## OPERATION –

### A. For patient weight

1. If display is not “On”, press On/Off (button “1”) to start scale operation.
2. Bring the bed to horizontal position using the CPR button on the handset (button 9). Check that the required bedding is on the bed, and the patient is not on the bed.
3. Master Zero – Press and hold Master Zero (button “3”) for 4 seconds and release after hearing two beeps.  
**Important Note** - Do Not activate Master Zero - button ‘3’ when patient is in bed.
4. Scale will read 0.0 (Zero) – patient can then be placed on the bed.
5. The weight of the patient will be displayed after a few seconds.
6. After scale is not active for 10 minutes it will display “SCALE”. To show weight again, press “Weigh” button “2” or “Hold” button “6”.
7. Kg/ Lb – to switch between Kg/Lb press “WEIGH” (button “2”) for 3 seconds.

### B. Adding / Removing bedding / items to the bed

1. Press “HOLD (button “6”).
2. Add items to the bed.
3. Press “WEIGH” (button “2”) and the weighing feature will resume, not changing the patient weight reading.
4. To remove items from the bed, follow same procedure.

(Turn to other side)