

# Total-LiftBed™ VG-TLB425V5 Operation Manual

(For detailed instructions, safety guidelines and warnings please refer to the owner's manual)

## Please Note:

When reaching the end of the movement of each function (unless stated differently) two beeps will be heard as confirmation. All Functions should be pressed continuously.

## Safety Note:

When the bed is in tilting position all other functions will not work except for (9) (11) & (12) (12 after unlock) for Tilting the bed up & Down to flat position. This is unless the bed is in Manual mode operation (see below)

## TILT UNLOCKING (Caregiver Only!)

Press Buttons (11) and (12) together for 3 seconds until two beeps will be heard.

**INITIALIZATION** – Buttons 1+2

**CLEAR ERRORS** – Buttons 3+4

## MANUAL MODE (Caregiver only)

Pressing buttons 5+6 for 3 seconds will create short beeps which indicates entering into Manual Mode, which enables to move each Actuator separately.

See image for actuator movement



Back rest

Leg rest

Hi-Low

Footboard

Trendelenburg

Tilt Down - Up

**Any time you have had to use Manual Mode to lower the Total Lift bed, the bed should be inspected by a qualified technician**

**Raising The Back Rest**



**Lowering The Back Rest**

**Raising The Leg Rest**



**Lowering The Leg Rest**

**Raising the Bed Horizontally**



**Lowering the Bed Horizontally**

**Seating with Legs Down**



**Foot Board Out**

**Wheels Position:**  
Lower the Bed and activate the Wheels



**Trendelenburg**

**CPR Position**  
Bed Flat



**Tilting the Bed to Standing Position**  
*(Should be Unlocked first)*

VitalGo

## LED on indications

>30° Backrest >30 degrees.



Clear error (3+4) is pressed correctly.



Mains Power On.



Battery over 50%, if blinking 20%-50%, if no light <20%



Casters down & ready for operation



Tilt function unlocked.