

Total life bed transport instructions

1. Bring Ramp to the right level on the dock so that it will not make damage the casters (wheels) or loadcell legs.



Total life bed transport instructions

2. Press caster (Wheels) button on the bed until you hear two beeps (bed middle/sub frame is all the way up) and the caster button light is showing lit green.



Total life bed transport instructions

3. Turn bed scale off (To preserve battery power during delivery).



Total life bed transport instructions

4. Bring bed into truck, position against the wall, place casters (wheels) on both sides in neutral.



Total life bed transport instructions

5. Attach head side strap (Around Trapeze Holder)



Total life bed transport instructions

6. Attach the Foot side Strap to frame as shown below.



Total life bed transport instructions

7. when strapping the bed against the wall and the side rails are getting bent in. Add a foam or hard rubber block against the wheels to create a separation



Total life bed transport instructions

8. If transporting bed far, on bumpy roads or just for the longevity of the caster (wheels) we recommend using transport boards under the sub frame.

