

Vital Go Total Lift Bed V5

Patient Set-up and Troubleshooting Quick Reference Guide

Pre-Patient Set-up Checklist

Step 1: Plug Bed In/Power Check

- Plug bed into a working outlet
- Check hand controller to ensure the light below plug icon is illuminated green

Step 2: Complete Initialization Sequence

- Press and Hold Buttons 1+2** (Raise/Lower Head) without interruption on hand pendant or head of bed side rail until bed completes all movements (finishes with legs elevated and footboard extending)
- Use **Button 11** (CPR/Flat) to return bed to **"Home"** position on stationary frame

Step 3: Zero the Bed Scale

- Press **Button 1** (Power Button) on the Scale Box to power scale on (screen should read "Scale")
 - Bed must be in **"Home"** position
- Place linens & necessary equipment on bed
- Zero Bed Scale by **Pressing and Holding Button 3** (Total Weight/Bed Zero) for 3 seconds on the Scale Box
- Zero Footboard Scale by **Pressing and Holding Button 5** (Footboard/FootBoard Zero) for 3 seconds on the Scale Box
 - Ensure footboard is extended beyond mattress using **Buttons 7 & 8**

Step 4: Power Mattress On (refer to mattress tip sheet)

- Ensure CPR valve located at the Head of Bed is in **Closed** position
- Set numeric pressure set to patients weight using reference chart on mattress panel
- Set-Therapeutic surface (e.g. AP & Fowler, 5 minutes, etc)
- Implement any institutional practice standard protocols

Step 5: Transfer Patient Onto Bed



Basic Verticalization/Tilting Procedures

Step 1: Secure Patient Using Total Lift Bed Straps

A. Standard placement vs alternative based on clinical judgement

B. Step 2: Unlock Tilt Features

A. On hand pendant, **Press and Hold Buttons 11+12** (CPR/Flat & Tilt) ~ 3 sec to unlock tilt feature (light in the unlocked icon will illuminate)

- a. The Total Lift Bed will relock after 90 seconds of non-use

B. Check for lines/cords and obstructions before tilting

Step 3: Tilt Patient

A. **Press Button 12** (Tilt) on hand pendant to tilt until desired angle is achieved

- a. All other features (except CPR/Flat) will be locked when tilted >30 degrees

B. **Press Button 11** (CPR/Flat) to reduce angle of tilt as needed or to return to “**Home**” position on completion of session

Emergency Lowering from Tilt (Requires Reset)

A. Use manual release located at the head of bed

- a. Use if patient is experiencing a medical emergency or bed “freezes” in tilt position and will not return to flat using **Button 11** “CPR/Flat”
- b. **Pull** the stopping pin and **Push/Hold** the red lever outwards
- c. Use **Button 9** “Wheels Down” to lower bed on to wheels to reset and return bed to normal function



Transport Tips

- Press **Button 9** on hand pendant to lower bed to wheels
 - Can also **Press and Hold** CPR + Lock buttons on head of bed side rails
- Check brake pedals at head of bed on both sides are in desired position before moving or transporting bed
 - Lock vs Neutral vs Directional Steer
- When the power drive system is on, the bed will not move manually
 - Use the thumb throttle to engage the motor for easy driving



Troubleshooting Quick Reference

- Bed freezing, **Blinking Blue LEDs** on the side-rails, and/or gear indicator light illuminated on hand pendant
 - Indicates a bed software stoppage to prevent a bed malfunction or damage
 - **Press and Hold Buttons 3+4** (Raise/Lower Legs) on the hand pendant or head of bed side rail until beeping stops (~10 beeps) to clear error
 - Complete Initialization Sequence as outlined in **Step 2 of Pre-Patient Set-up** to resolve
- Low Pressure in Mattress Alarm
 - Check that CPR Valve at Head of Bed is in closed position and check for leakage at connections of air hoses at both pump and mattress
- For all other issues/ongoing problems contact your local distributor for further diagnostic and exchange